



C-celia™ C-Section
Skills Training Simulator

Care, Maintenance and Operating Instructions Manual

www.operativeexperience.com

410-287-5748

C-Section Skills Training Simulator

Table of Contents

Introduction.....	1
Mobius Retractor Information.....	1
Parts List	2
Uterine Incision	3,4
Abdominal Incision	4
Abdominal Repair	5
Vertical and Transverse Repair Uteri	6
Disassembly, Cleaning and Storage	7
Technical Support and Disclosures	8

{image}

[note: this needs to be retouched]

https://www.dropbox.com/s/fmjnd93oq1ev0uq/IMG_9051.JPG?dl=0


Welcome.

Congratulations on your purchase of the C-Section Skills Training Simulator by Operative Experience. Operative Experience is a leading manufacturer of patient simulators that fully operable and feature incredibly accurate and tactile anatomy, realistic blood and fluids, and are uniquely designed to provide learners with a high-adrenaline, real-world operative experience.

This user guide provides information on the system's use and capabilities, step-by-step procedures for set-up and operation and important information regarding maintenance and storage of your simulator.


Parts List

- | | | |
|-------------------------------------|------------------------------------|-------------------------------|
| A. Back Plate | B. Simulator Base | C. Uterine / pelvic lock |
| D. Screw driver | E. Screws | F. Intestines |
| G. Vertical Repair Uterus | H. Transverse Repair Uterus | I. Soft Abdomen |
| J. Abdominal Repair Platform | K. Pregnant Abd. Incision Platform | L. Uterine Incision Platform |
| M. Abdominal Incision/Repair Insert | N. Uterine Incision Insert | O. Uterine Amniotic Fluid Bag |

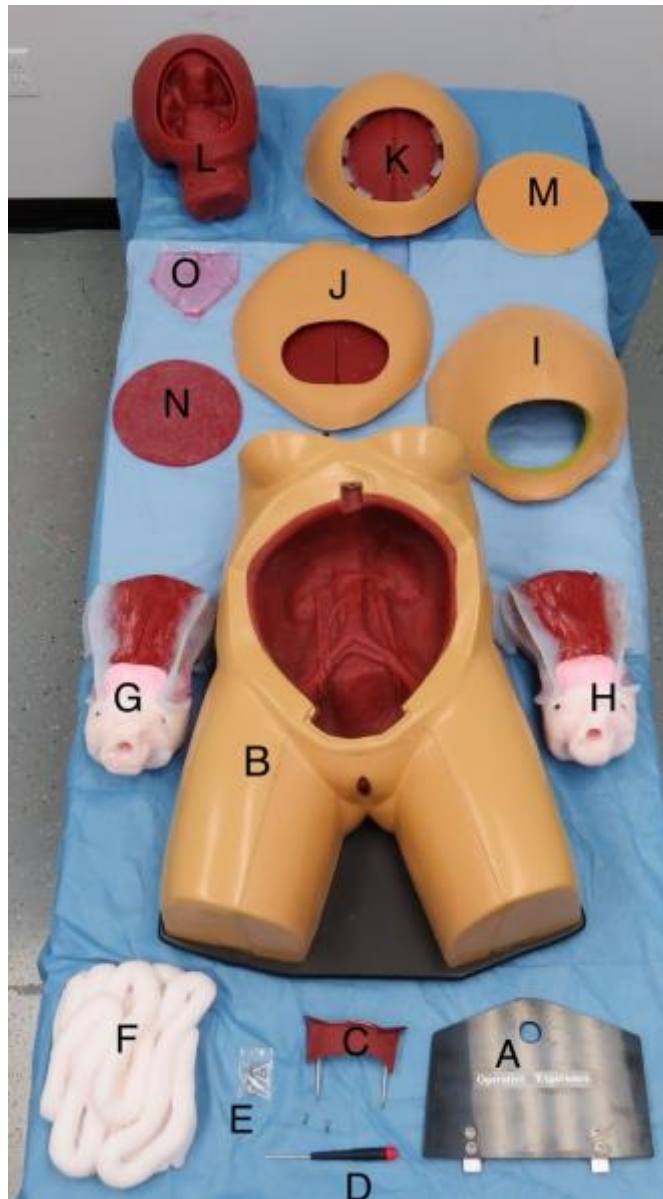

Uterine Incision

Amniotic fluid bag placement

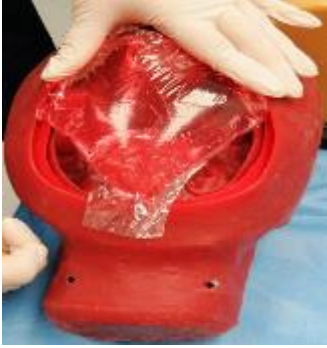

Align back of uterus with score lines and arrow.



Place uterus insert over fluid Bag with arrow on back pointing up and center.



Begin tucking insert over ridge on plastic ring starting from bottom to top.....



....side to side



Finish tucking all around as shown



Place platform into simulator Base



Insert uterine/pelvic lock (optional) with 2 screws (do not over tighten)



Ready for use



(C-Section Skills Simulator)

3

Uterine Incision Cont.

Optional – abdominal placement



Hook into place using attached Hooks underneath abdomen



Insert locking pin into abdominal ring.



Abdominal Incision

Image of Abdomen incision platform with insert



Begin inserting into platform



Tuck into place as shown, assuring metal guides on back of insert are clipped over tabs on platform



Place platform on simulator base.



Insert locking pin into abdominal ring



Ready for use



(C-Section Skills Simulator)

Abdominal Repair

Image of abdominal repair platform with incised abdominal insert.



Place insert into repair platform



Place platform into base



Insert locking pin into abdominal ring



Ready for use



(C-Section Skills Simulator)

Vertical and Transverse Repair Uteri

Image of vertical and transverse
Repair accessories



Insert uterus into simulator base



Place intestines behind uterus



Insert uterine/pelvic lock and
(2) screws (screws optional if
using abdomen) DO NOT OVER TIGHTEN



Ready for use



OPTIONAL : soft abdomen in place



Uterus delivered, ready for use



*** Note: Abdominal incision platform may also be used in conjunction with either uterus incision platform or repairable uteri.**

Mobius Retractor

The use of a Mobius Elastic Retractor, while not required or provided, will extend the life of the model.



(C-Section Skills Simulator)

6

Disassembly, Cleaning and Storage

Cleaning:

Model can be cleaned with water or water with a mild soap. DO NOT store model overnight without washing, and wash and dry thoroughly before storage.

Note: The entire model can be “hosed down”

Storage:

Surface areas, including the inside compartment, should be cleaned with mold soap and water and dried before storage.

Technical Support:

*** Simulators must be serviced by qualified OEI technicians only***

OEI Office:	410-287-5748 x 302
Chief Engineer:	443-373-6815
Chief Operating Officer:	443-691-5179
Chief Executive Officer, Robert Buckman, Jr., M.D.	484-557-8876

www.operativeexperience.com

*** This Manual includes confidential, unpublished property of Operative Experience Inc.
Photocopying and distribution are strictly prohibited. All Rights Reserved 2014.**

Disclosure

To prevent electronic and mechanical system failures, please comply with the instructions included in this manual.

Infuse manikin with supplied or approved simulated blood and fluids only.

*** FAILURE TO COMPLY COULD RESULT IN CATASTROPHIC SYSTEM FAILURE***

(C-Section Skills Simulator)

