

Episiotomy Suture Training Module P96

EPISIOTOMY SUTURE TRAINING MODULE P96

> INTRODUCTION

You have purchased an Episiotomy Suture Training Module P96 which will upgrade your Birthing Simulator P90 PRO or BASIC (1020333 -1020332) with realistic postpartum episiotomy suturing and repair skills trainer inserts. The soft skin will support real sutures and will enable the trainee to gain experience in instrument handling as well as suture techniques and tying knots.



> DELIVERY CONTENT

- 1 module base
- 2 episiotomy suture inserts, pre-cut
- 1 episiotomy suture insert, uncut
- Safety plate with 4 screws
(for the optional insert in Birth Simulator P90)

> FEATURES

- Compatible with Birthing Simulator P90 PRO and Basic
- Easy installation of the different inserts and of the module in the Birthing Simulator P90
- Two inserts feature medial episiotomy and mediolateral episiotomy with tears in the labia minora
- One insert is uncut and enables the trainee to choose the episiotomy location he wants to train
- Realistic soft skin
- Supports the use of real suture instrument
- Supports real sutures
- Durable reusable skin
- Made of latex-free material

EPISIOTOMY SUTURE TRAINING MODULE P96

> ASSEMBLY

- If used in conjunction with the Birthing Simulator P90 (Basic and PRO), the module can be put into the body of the birthing simulator. Place the rubber feet of the birthing simulator on a non-slip surface such as a table.
- Remove the entire abdominal cover of the Birthing Simulator P90 (Basic and PRO).
- Remove the cervix and the birth canal insert of the birthing simulator P90 (only in PRO version).
- Loosen and remove the 4 screws indicated on Fig. 2 to place a second time the Safety plate with the 4 integrated screws (which remain on the safety plate) (Fig. 1).
- Attach the safety plate (Fig. 1) with the 4 integrated screws (which remain on the safety plate) (Fig. 2).

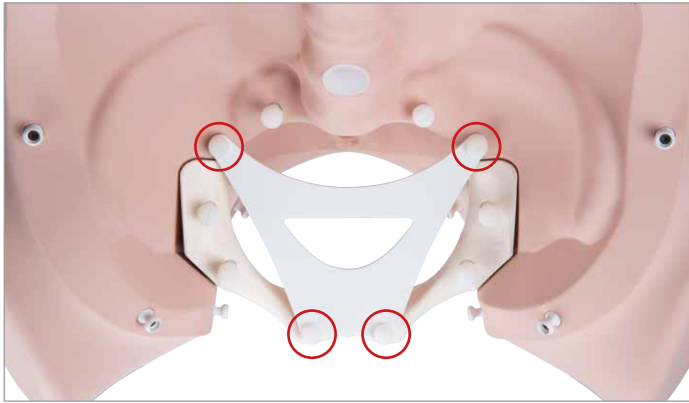


Fig. 2

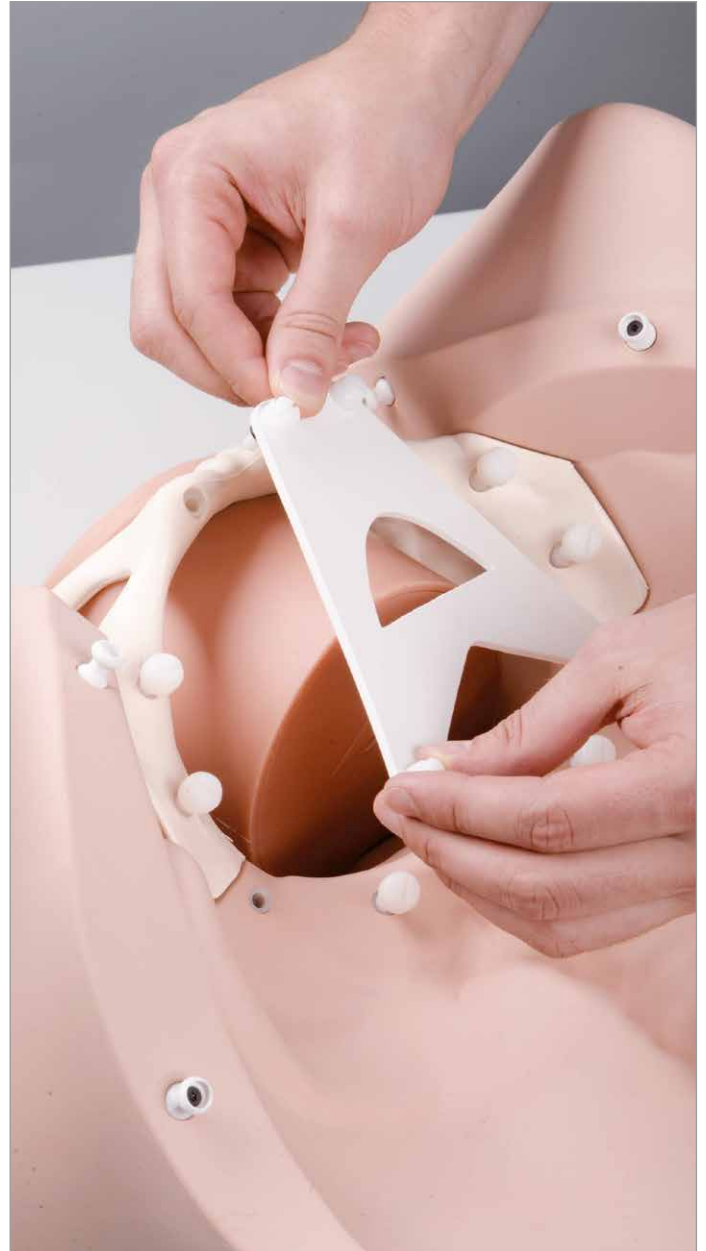


Fig. 1

> BASIC VERSION ONLY:

- Detach the rear phantom closure (connected with the white pins) from the abdominal cover (this can be used as a cover in normal use to prevent the lubricant running into the recess) (Fig. 3).
- Remove the vulva insert from the abdominal cover.
- Attach the abdominal cover (only around the coccyx) (Fig. 4).
- Insert the episiotomy module from underneath into the pelvis.
- Ensure that the insert has been clicked firmly into the basic body via the phantom closure.
- Close the abdominal cover completely (Fig. 5).

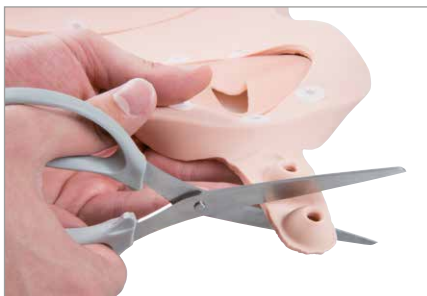


Fig. 3



Fig. 4



Fig. 5

EPISIOTOMY SUTURE TRAINING MODULE P96

> CLEANING AND CARE

The surface of the trainer can be cleaned with a damp cloth, or rinsed under running water. Please ensure that all parts are completely dry before storing.

> TECHNICAL DATA

Dimensions:

Simulator (H x W x D): 14 x 24 x 18 cm

Weight: approx. 1 Kg

Operating conditions:

Operating temperature: 0°C bis +30°C

Storage temperature: -10°C bis +40°C

> CONTACT



3B Scientific

A worldwide group of companies

3B Scientific GmbH

Rudorffweg 8 • 21031 Hamburg • Germany

Phone: + 49 (0)40-73966-599 • Fax: + 49 (0)40-73966-598

3bscientific.com • naturalsciences@3bscientific.com

> ORDER LIST

Item No.	Replacement Parts and Consumables
----------	-----------------------------------

1022214	Replacement set Episiotomy suture training pre-cut module P96
---------	---



1022215	Replacement set Episiotomy suture training module P96
---------	---



Item No.	Also recommended for the midwifery and obstetrics training
----------	--

1020332	P90B Birth Simulator BASIC
---------	----------------------------

1020333	P90P Birth Simulator PRO
---------	--------------------------

1020628	Birth Stages Trainer
---------	----------------------

All trainers, simulators and replacement parts available at 3bscientific.com