


# VT SELECT™

## Manual Resuscitator with Select Valve


The VT Select is a manual resuscitator with a timed refill valve. It is recommended by the AHA to deliver 10 BPM of 500-600mL to a standard adult. When activated, the valve restricts the refill of the bag to ~4 seconds. This provides your patient ~10 BPM, helping to eliminate hyperventilation of the patient. When the valve is activated, press your thumb and three fingers simultaneously together utilizing the finger ridges to deliver approximately 500-600mL to the patient. Additionally, the BVM is smaller (1200mL) than most adult bags on the market. This helps to reduce the risk of hyperinflation or barotrauma. The VT Select can be supplied with an optional PEEP valve, Filter, and dial manometer.

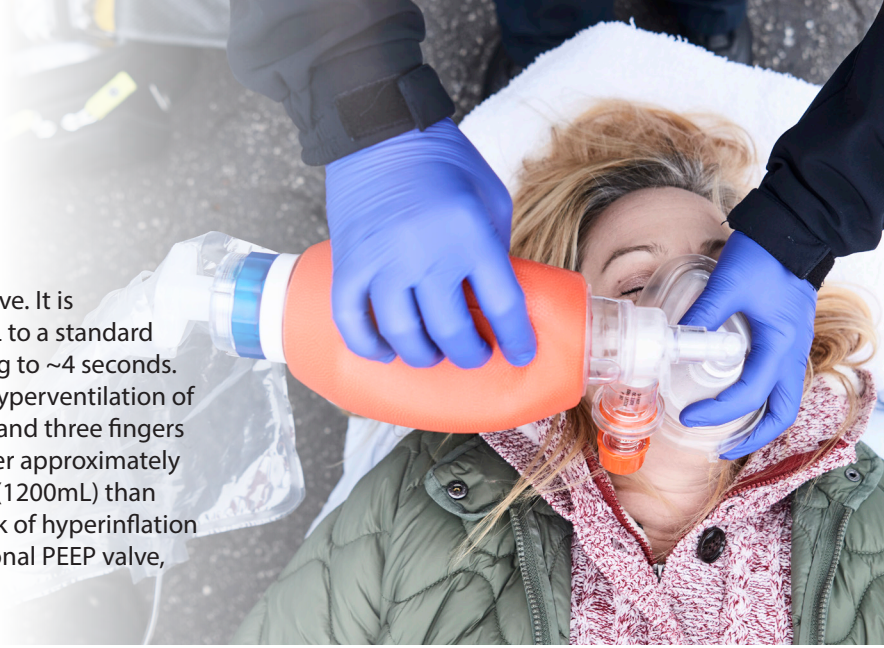

1200cc Bag Volume

Timed refill delivery (~10 BPM)


1-Hand Technique

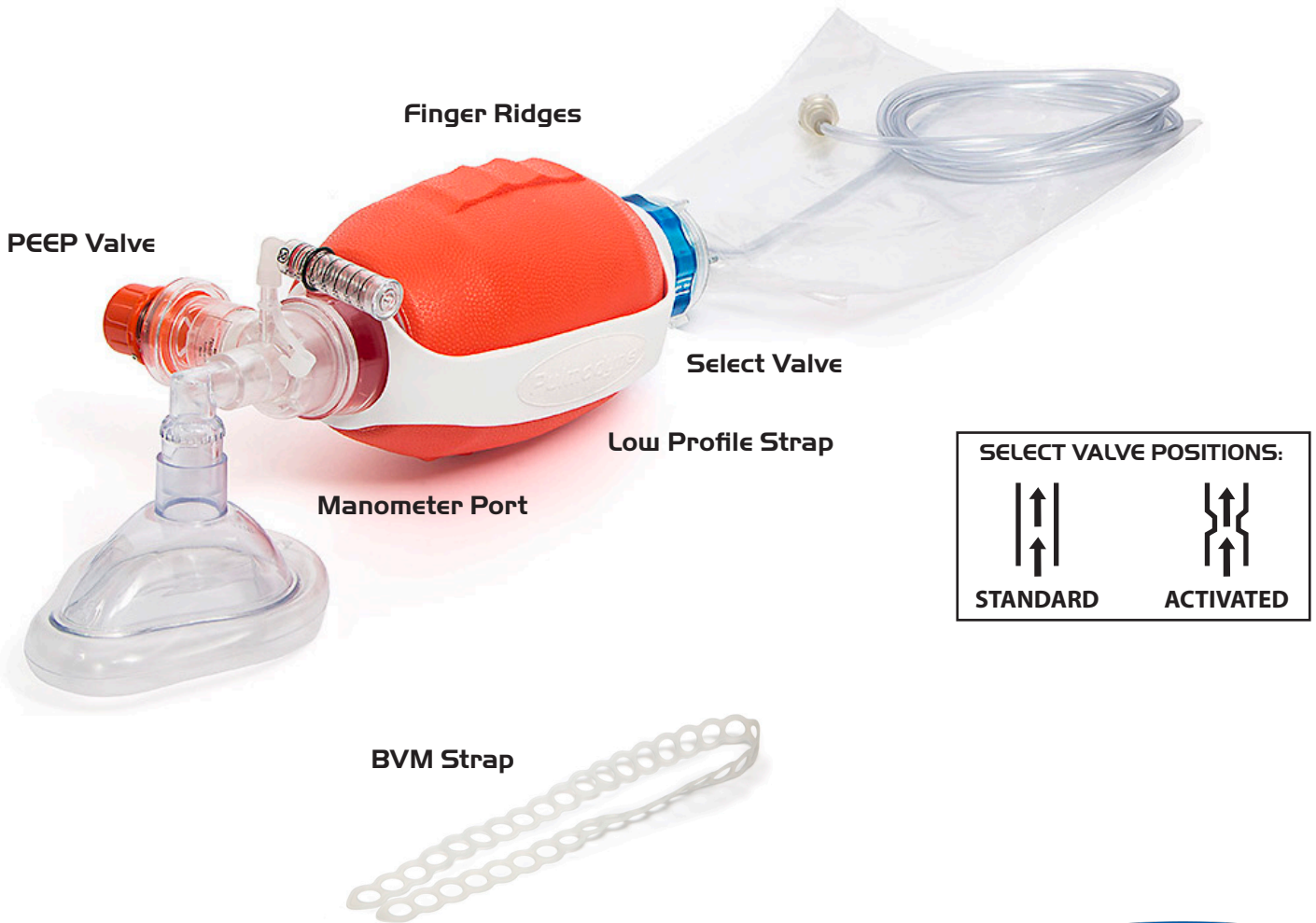

2011R-07-25-2022

**Optional Accessories:**

- Adjustable PEEP Valve
- Dial Manometer
- Filter

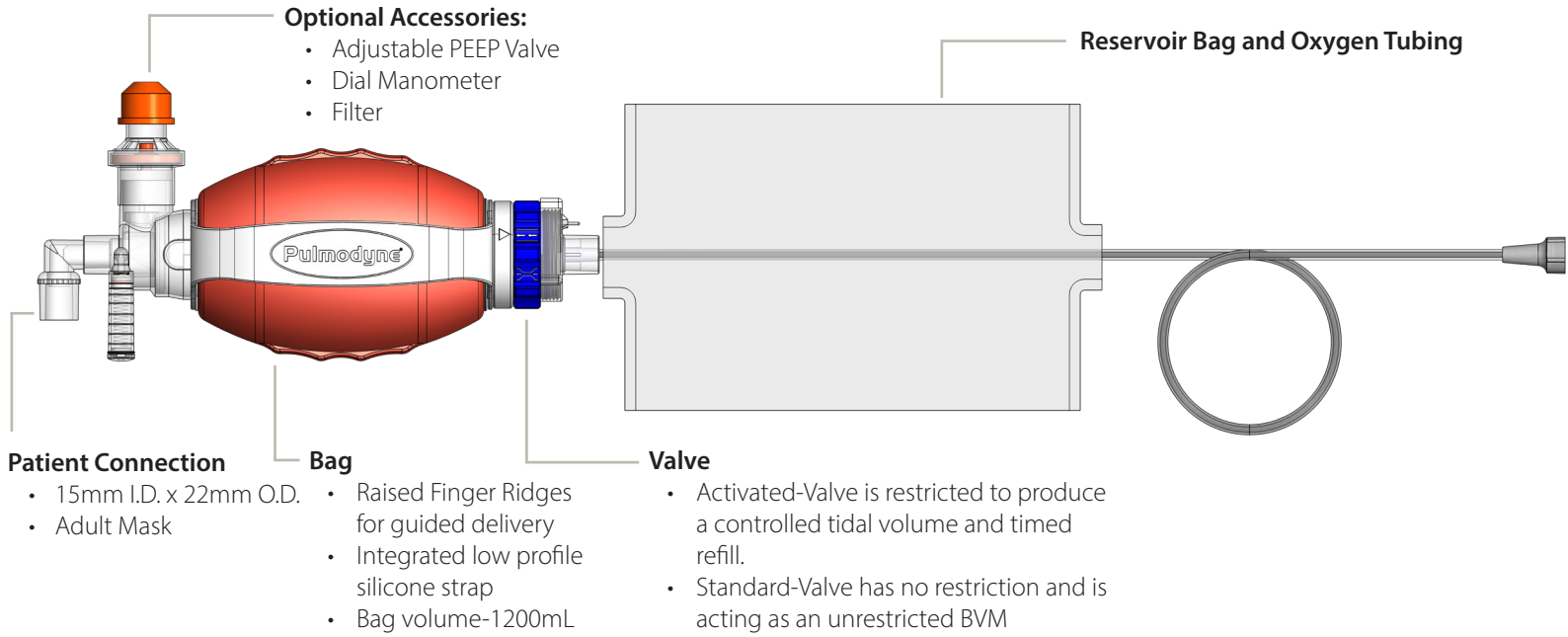

**Patient Connection**

- 15mm I.D. x 22mm O.D.
- Adult Mask

**Bag**

- Raised Finger Ridges for guided delivery
- Integrated low profile silicone strap
- Bag volume-1200mL

**Valve**

- Activated-Valve is restricted to produce a controlled tidal volume and timed refill.
- Standard-Valve has no restriction and is acting as an unrestricted BVM

**Specifications:**

Body Mass Range	>40kg (Adult)	Maximum Bag Volume	1200mL
Storage Temperature Range	-40° C to 60° C	Delivered Volume Range	(1 hand) 530mL (2 hand) 690mL
Operating Temperature Range	-18° C to 50° C	Patient Connection	15mm I.D. x 22mm O.D.
Inspiratory Resistance	< 5 cm H <sub>2</sub> O (0.5kPa) @ 50LPM	Exhalation Port	30mm
Expiratory Resistance	< 5 cm H <sub>2</sub> O (0.5kPa) @ 50LPM		

As demonstrated in previous studies, trained clinicians tend to hyperventilate and hyperinflate their patients suffering from cardiac arrest. Manual resuscitation is highly variable from provider to provider, and the VT Select helps to minimize the variability in treatment by utilizing a timed refill mechanism.

**VT Select™ - Manual Resuscitator with Select Valve**

Part No.	Description	Case/Qty.
313-5590	VT Select BVM with Manometer, PEEP Valve, Filter - Adult Mask	8
313-5591	VT Select BVM with Manometer, PEEP Valve - Adult Mask	8
313-5592	VT Select BVM with PEEP Valve - Adult Mask	8
313-5593	VT Select BVM with Manometer - Adult Mask	8
313-5594	VT Select BVM- Adult Mask	8

**VT Select™ - Accessories**

Part No.	Description	Case/Qty.
313-5560	Angled Manometer	10
313-5584	Standalone Adjustable PEEP Valve BVM (22mmF)	10
313-7145	BVM Strap	10